



# Folding Wooden Bed Tray User Instructions

Manufactured for and distributed in the UK by:  
Independent Living Aids Ltd, H300 Edison Road, Hams Hall, Coleshill, Birmingham B46 1AB

For more products from the Mobilease daily living aids range,  
please refer to the Mobilease leaflet or visit our website  
[www.mobilease.co.uk](http://www.mobilease.co.uk)

Please check the website for updated instruction manuals

After Sales Support:	
UK / N. IRELAND HELPLINE NO.	00800 45672245
WEB SUPPORT	<a href="http://www.mobilease.co.uk">www.mobilease.co.uk</a>



Prior to use please read all instructions carefully.  
Keep instructions in a safe place for future reference.

Independent Living Aids Ltd is a member of the British Healthcare Trades Association (BHTA)

Version 3.0



## Mobilease Folding Wooden Bed Tray

This attractive wooden bed tray provides a base for reading and writing and lends itself for many other uses around the home.

It is easily wiped clean, folds flat for easy storage and can be used at either an angled or flat settings.



### Using the Mobilease Folding Wooden Bed Tray

Remove all packaging from the bed tray, unfold both wooden legs and stand tray on a flat even surface.

To close, push both wooden legs back up inside the base of the tray.

To change the angle of the tray, lift the top of the tray upwards and locate the metal bracket fixed to the underside of the tray. Select the required angle and slot the metal bracket into the corresponding groove; lower the top of the tray to rest. Always ensure that the metal bracket is securely resting inside the selected groove.



### Safety and Maintenance Instructions

Care should be taken not to overload the Mobilease Bed Tray.

This bed tray is not suitable for use as a carrying tray and care should be taken not to apply excess weight to the tray.

Take care when handling the bed tray and hold edge of tray to avoid trapping fingers.

It is not recommended that this bed tray be used as a base for hot drinks or food.

### Cleaning Instructions

To clean wipe with a damp cloth. Always allow to dry thoroughly.

Do not immerse in water.

Store in a dry place.

### Technical Data

Length: 495mm (19.5") approx

Width: 310mm (12.25") approx