



### Cleaning Instructions

All parts can be cleaned with mild soapy water including the adjusting screws which can be popped out of the brackets for cleaning.

### Technical Data

Max. User Weight/Loading: 18 3/4st (120kg)

Manufactured for and distributed in the UK by:  
Glopac Mobility Ltd., Edison Road, Hams Hall, Coleshill, Birmingham B46 1AB

**For more products from the Mobilease daily living aids range,  
please visit our website [www.mobilease.co.uk](http://www.mobilease.co.uk)**

#### After Sales Support:

**UK / N. IRELAND HELPLINE NO.  
WEB SUPPORT**

**00800 45672245  
[www.mobilease.co.uk](http://www.mobilease.co.uk)**



Glopac Mobility Ltd. is a member of the British Healthcare Trades Association (BHTA)



Version 1.0



# Raised Toilet Seat User Instructions



A Mobilease quality  
care product

**Prior to use please read all instructions carefully.  
Keep instructions in a safe place for future reference.  
We recommend that this equipment be installed by a  
competent person.**

# Mobilease Raised Toilet Seat

**The Mobilease raised toilet seat is designed to provide assistance with toileting. It fits most standard toilet seats raising their height by 4" (100mm). The comfortable contoured seat allows for even weight distribution and its wide frontal opening facilitates personal cleaning.**

**The Mobilease raised toilet seat secures firmly onto the existing toilet seat with adjustable side brackets. Fast and simple 'no tool' installation.**



## Fitting instructions

The Mobilease raised toilet seat is fitted to the existing toilet seat. When correctly fitted, the Mobilease toilet seat will raise the existing toilet seat height by 4" and will fit the majority of standard toilet seats.

Prior to installation ensure that the existing toilet seat assembly is fitted securely and undamaged. Lift the lid of the existing toilet seat prior to installation of the Mobilease toilet seat.

1. Place the Mobilease raised toilet seat with its top down onto a flat surface.
2. Slip one of the L-shaped brackets into one of the slots in the side of the Mobilease raised toilet seat (see A). Then push lightly on the bracket and turn the knob clockwise until its thread engages. Repeat with the second bracket on the opposite side of the seat.
3. Turn the knobs until the brackets are farther apart than the width of the toilet bowl.
4. Turn the Mobilease raised toilet seat over and place it onto the existing standard toilet bowl. Note that the end with the narrow opening should be placed at the front end and the end with the brackets near the back of the existing toilet seat.
5. Tighten both brackets until snug against the existing toilet bowl.
6. See Image B for a securely and correctly fitted Mobilease raised toilet seat.



## Safety and Maintenance Instructions

The Mobilease raised toilet seat is not designed to support user weights above 18 3/4st (120kg).

Always ensure the Mobilease raised toilet seat is securely installed and fitted to the existing toilet by a competent person prior to use.

Do not hold onto the seat as a support when getting on and off the toilet seat.

Do not sit on top of the lid of the Mobilease raised toilet seat.

Inspect regularly for wear and damage as well as for secure fit.