



If the chair is being used by multiple users for reasons of safety and hygiene, ensure all parts of the chair are checked and thoroughly cleaned down following the cleaning instructions after each use.

#### Technical Data

Length: 850mm (33.5") approx

Height: 1020mm (40") approx

Width: 550mm (21.5") approx

Armrest Height: 760mm (30") approx

Seat Depth: 440mm (17.5") approx

Seat Width: 520mm (20.5") approx

Seat Height: 520mm (20") approx

Max. User Weight: 114kg (18st)

Manufactured for and distributed in the UK by:  
Independent Living Aids Ltd, Edison Road, Hams Hall, Coleshill, Birmingham B46 1AB

**For more products from the Mobilease daily living aids range,  
please refer to the Mobilease leaflet or visit our website  
[www.mobilease.co.uk](http://www.mobilease.co.uk)**

**Please check the website for updated instruction manuals**

#### After Sales Support:

**UK / N. IRELAND HELPLINE NO.**

**00800 45672245**

**WEB SUPPORT**

[www.mobilease.co.uk](http://www.mobilease.co.uk)

**BATCH NUMBER**

**DATE OF PURCHASE**

Independent Living Aids Ltd is a member of the British Healthcare Trades Association (BHTA)

Version 2.0



## Transport Chair User Instructions



A Mobilease quality  
care product

**Prior to use please read all instructions carefully.  
Keep instructions in a safe place for future reference.  
Please make a note of your batch number in the designated space on  
the back page of this instruction sheet for future reference.  
The batch number can be found on the product label located on one of  
the vertical metal struts.**

# Mobilease Transport Chair

**The Mobilease Transport Chair is designed for occasional short-distance use on even terrain such as transit to and from a car, shopping centres, airports, etc. The chair is intended to be 'attendant operated' and requires a fully mobile adult to be in attendance at all times when the chair is occupied.**

Of lightweight steel construction this weather-resistant transport chair features detachable handles, fold-up footrests and a carry bag. Easy assembly without any tools being required.

**It is not intended to be used as a traditional wheelchair.**

## User Instructions

### Assembling the chair

Remove the chair from its box and stand on its wheels.

To open and unfold the chair, stand behind it and pull the backrest bars apart applying downward pressure on the seat as necessary to ensure the chair is fully opened.

Ensure the detachable handles are securely located with the handles facing to the rear of the chair. Now fold down the leg supports and foot rests fully.

Before using the chair, check that the dual-wheel brake locks are fully engaged against the tyres on both rear wheels and that the front wheels are facing forward with the foot plates folded out of the way to aid transfer in and out of the chair.



Do not use the chair until the occupant is sitting comfortably and not leaning forward

### Disassembling the Chair

To fold the chair after use, fold up leg supports and foot rests. Now standing behind the chair, push the seat upholstery upwards from underneath and then proceed to push the back bars and front arms towards each other in a concertina action to fold the frame. Place the chair in its carry bag for safe storage. Keep fingers clear of the chair's arm supports and folding mechanism.

### Packing in to the carry bag

Remove the handles and insert the transport chair wheels first into the carry bag, then pull the drawstring tight to close up the opening.

### Safety Instructions

Before using the chair always ensure that the dual-wheel brakes are fully locked against the tyres of both rear wheels. The front wheels must be forward-facing with the footplates folded out of the way as not to interfere with transfer in and out of the chair.

Check that the metal locking pins securing the detachable handles are fully engaged.

Never stand on the footplates or on the seat of the chair.

Do not lean forward in the chair whilst moving as this could cause the chair to tip.

Do not use on uneven surfaces

Safe use and operation of the transport chair is of major importance at all times by the user as well as by the attendant. The attendant must ensure they are familiar with the proper assisting procedures in any situation that requires the use of this chair.

If there are any doubts about the suitability of this chair for an individual, seek advice from their healthcare professional.

Not suitable for children

### Cleaning & Maintenance Instructions

Proper regular care and maintenance is required to keep this transport chair in good working condition.

Inspect the chair regularly with particular attention being paid to the wheels, brakes and brake pads as well as to the upholstery of the chair.

The chair should not be used if the brakes, brake pads, wheels or the upholstery show any signs of wear and tear or damage.

After each use, wipe off the frame using a clean cloth.

If the chair is exposed to moisture dry chair immediately after use.

Spray polish can be used on metal parts.

Clean the tyres regularly using a damp cloth and remove any grit or small particles that have become trapped in the tyres.

The upholstery can be wiped down with mild soapy water and then wiped dry.