



# Unisex Urine Bottle

## User Instructions

Manufactured for and distributed in the UK by:  
Independent Living Aids Ltd, H300 Edison Road, Hams Hall, Coleshill,  
Birmingham B46 1AB

For more products from the Mobilease daily living aids range,  
please refer to the Mobilease leaflet or visit our website  
[www.mobilease.co.uk](http://www.mobilease.co.uk)

Please check the website for updated instruction manuals.

**After Sales Support:**

UK / N. IRELAND HELPLINE NO.  
WEB SUPPORT

00800 45672245  
[www.mobilease.co.uk](http://www.mobilease.co.uk)



  
A Mobilease quality  
care product

Independent Living Aids Ltd is a member of the British Healthcare Trades Association (BHTA)

Version 3.0



Prior to use please read all instructions carefully.  
Keep instructions in a safe place for future reference.

## Mobilease Male or Female Urine Bottle

This hygienic urinal with easy-fit female adaptor is ideal for travelling or bedside use.

The 1 litre capacity plastic container is securely closed by a snap-on plastic cap to reduce odours and prevent spillage. Use of the urine bottle is made easy by a large carry handle and graduated markings for measuring.



### Using the Mobilease Urine Bottle

Prior to use always check the urine bottle for damage such as cracks, etc. Do not use if damaged.

Remove the snap-on cap prior to use and close firmly after use.

For female users, remove the snap-on cap and place the enclosed adaptor firmly inside the neck of the bottle.

### Safety and Maintenance Instructions

Always ensure the snap-on cap is securely placed over the neck of the urine bottle to prevent odour and spillages.

Regularly check for damage.

### Cleaning Instructions

Empty then clean and rinse the urine bottle after each use with mild soapy water to prevent odours.

### Technical Data

Capacity: 1 litre